**GB** English – instructions  
NRX® Strap

**General information**

Read the instruction carefully. Patients with diabetes, circulatory problems or sensitive skin should consult a doctor before use. If you suffer from allergy, check the material content before use.

**Usage and function**

The NRX® strap has been developed to be used to provide a light joint support. It can be cut in both length and width without fraying. The strap can be applied over the joint and the secured with the included hook tabs. The NRX® strap provides increased proprioception, local heat insulation and individual compression. The strap can be used in water without any problems.

The NRX® strap is available in 30 mm, 50 mm and 110 mm widths. Additional hook pieces can be purchased separately.

**Material content**

The strap is made from 100% CR-neoprene covered with a textile made from nylon and spandex. The hook is made from polypropylene. The NRX® strap is 100% latex free.

**Caring instructions and precautions**

We recommend to machine wash the strap once a week in 60 degrees, using a laundry bag. Remove the hook parts before washing. Patients with sensitive skin, diabetes or poor circulation should consult a doctor before use. Patients with allergies should check the material content before use.

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© Copyright MediRoyal Nordic AB, Sweden · 2017 · 25086DR · [www.vajjer.se](http://www.vajjer.se)